





## INSTALLING THE BIOCAM2.0™2.0 ON WINDOWS 7 (32-bit and 64-bit)

1. **Download** The BioCam2.0™2.0 for windows 7 form our website [www.goldlineBioCam2.0™.com](http://www.goldlineBioCam2.0™.com) or [www.BioCam2.0™vet.com](http://www.BioCam2.0™vet.com)

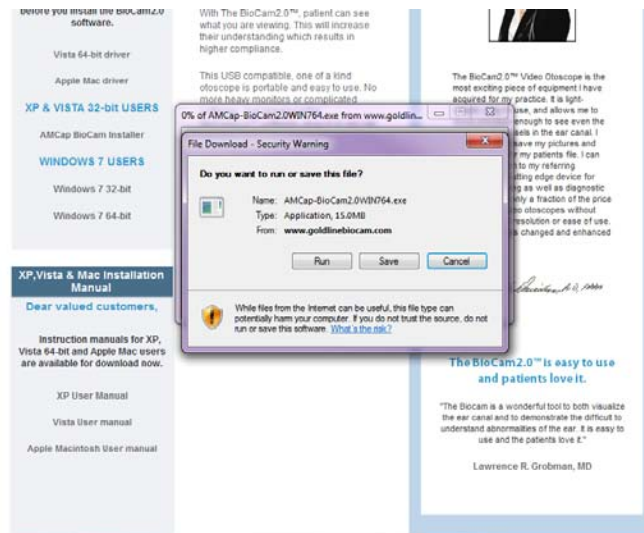


2. **Double Click** on the **installer** to start installation.
3. **User Account Control window** will appear r asking" Do you want to allow the following program from an unknown publisher to make changes to this computer?" **Select Yes** to continue with installation.


4. AmCap- BioCam2.0™2.0 - Install Shield Wizard will appear . **Select Next.**

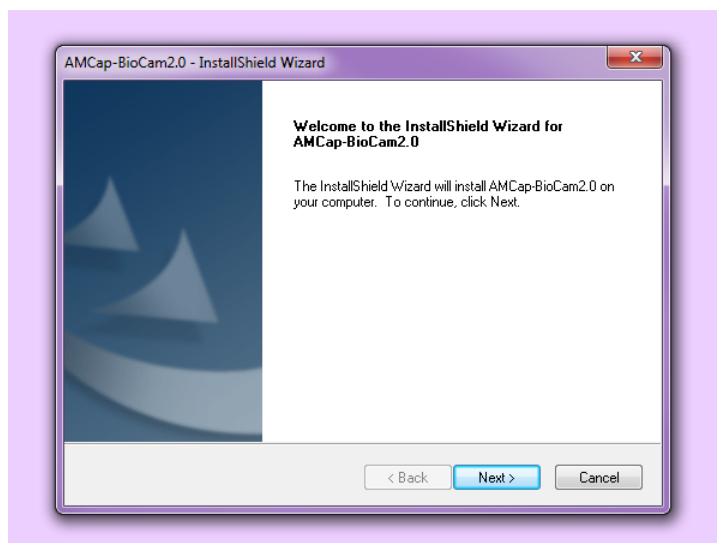


5. Choose destination Location. Destination folder should be in c:\ Program Files (x86)\BioCam2.0™\AmCap-BioCam2.0™2.0\ . **Select Next** to continue with installation.

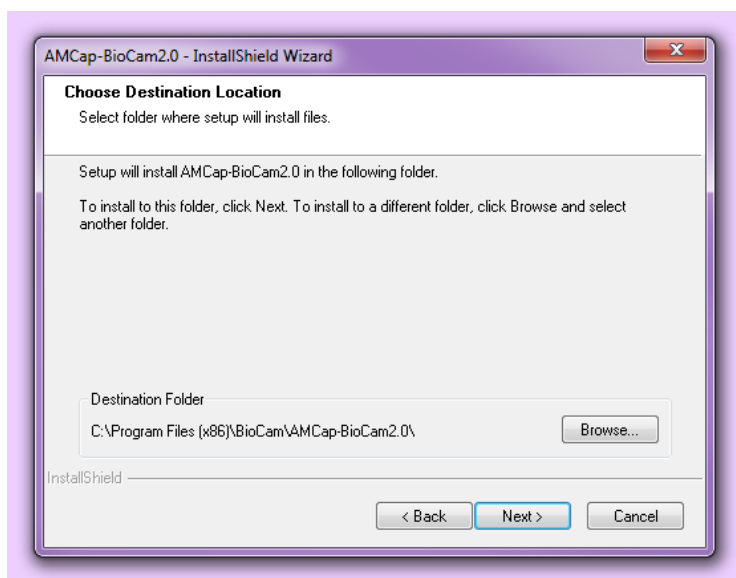


6. **Select next** when the Install Shield Wizard prompt to install the BioCam2.0™2.0 on your computer.

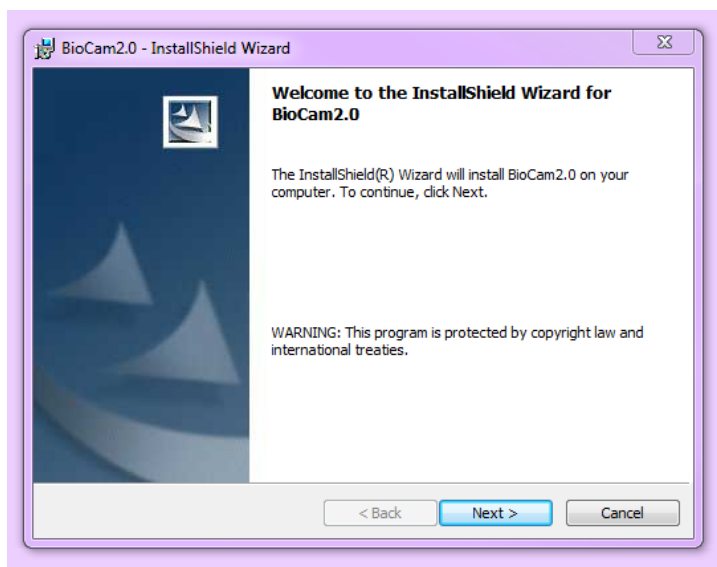


7. **Click Install** when the Install Shield Wizard is ready to install the program.

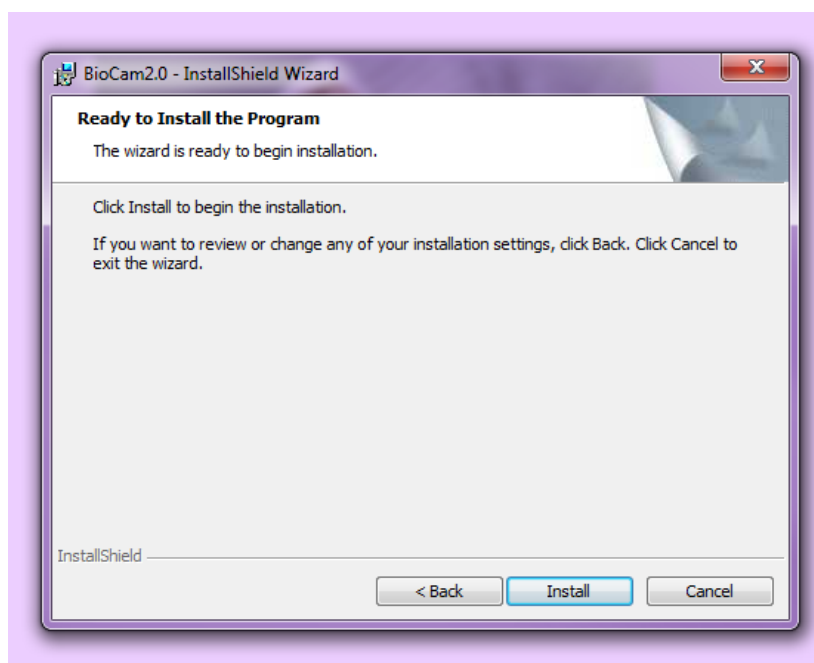


8. **Select Finish.**

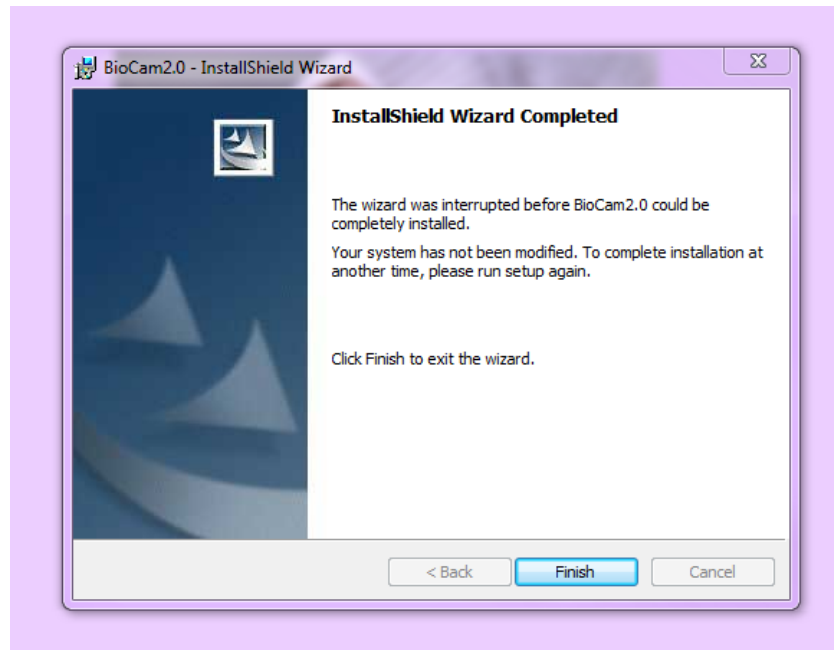

9. Choose the option "**Yes, I want to restart my computer now**". Click **Finish** to reboot your system.

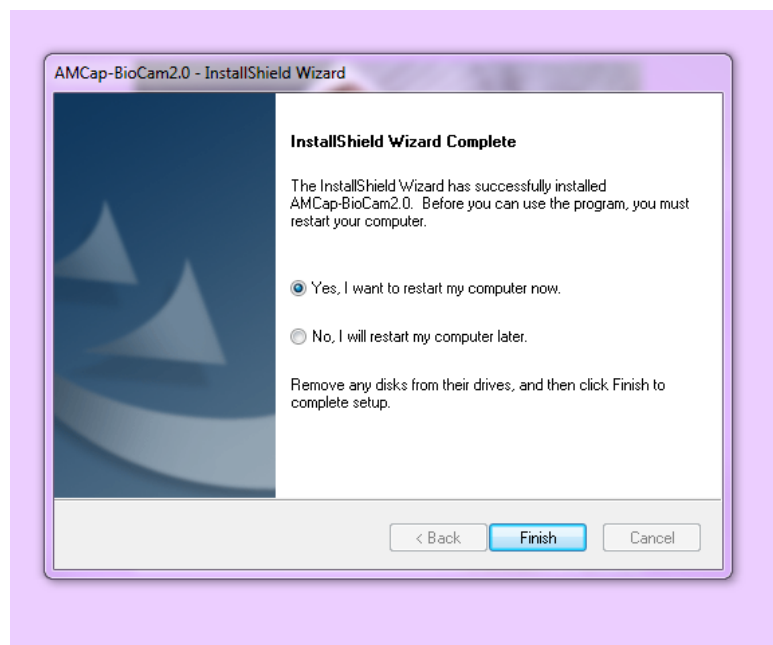

10. Once your computer is rebooted, **plug in** your BioCam2.0™2.0 video otoscope onto your computer USB port.

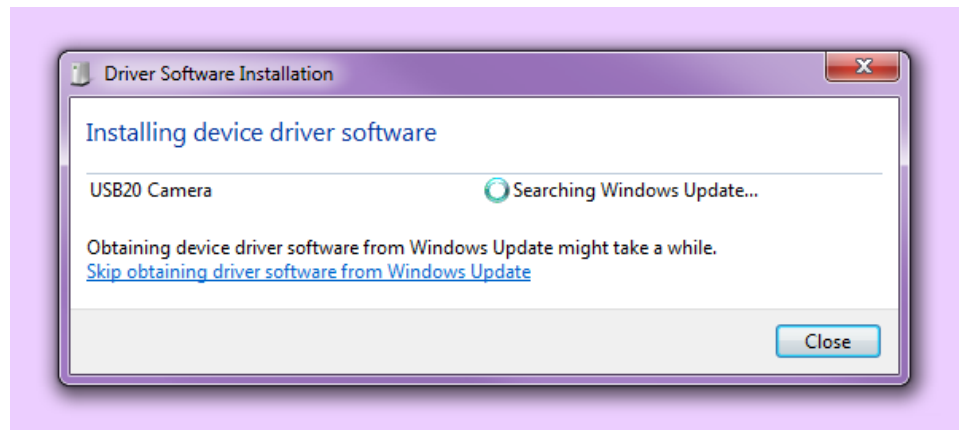

11. Wait for the system to install the driver. Once it is completed, your BioCam2.0™ is ready for usage.
12. **The Change the permissions folder that the pictures videos( CAPTURE FOLDER) get saved to, and make sure that the user has rights to write.**